Implantes mamários GC Aesthetics®

10 anos

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Prospective and multicentric long-term proven safety data
Duteille F, et al. Aesthet Surg J Open Forum. 2019;1(2):ojz012.

GC Aesthetics®

A Confident Choice for Life™

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ORIGINAL ARTICLE

Breast

Five-year Safety Data for Eurosilicone's Round and Anatomical Silicone Gel Breast Implants

Franck Duteille, MD*
Michel Rouif, MD†
Sophie Laurent‡
Máirín Cannon, PhD§

Background: Multicenter prospective studies assessing the safety and efficacy of silicone gel breast implants are relatively rare. Eurosilicone S.A.S. present their safety and efficacy data herein for the largest European silicone gel breast implant study published to date.

Breast Surgery

Eight-Year Safety Data for Round and Anatomical Silicone Gel Breast Implants

Franck Duteille, MD; Pierre Perrot, MD;
Marie-Hélène Bacheley, MSc; and Sharon Stewart, PhD

Aesthetic Surgery Journal
2018, Vol 38(2) 151-161
© 2017 The American Society for
Aesthetic Plastic Surgery, Inc.
Reprints and permissions:
journals.permissions@oup.com
DOI: 10.1093/asj/sj117
www.aestheticsurgeryjournal.com

OXFORD
UNIVERSITY PRESS

Downloaded from https://

Breast implants are the lead by volume by women techniques such as

From the *Centre Nivolas, France; †Eurosilicone S.A.S., 71 and ‡GC Aesthetics; Received for publication February 11, 2019; Copyright © 2019 Williams & Wilkins, a division of American Society of Plastic Surgeons; article distributed under Attribution-Non

Abstract
Background: Undertaking long-term implant surgery.
Objectives: The Paragel breast implants.
Methods: In this study, 8 years. Descriptive statistics.
Results: Capsules (Baker Grade II/IV) were identified by surgeon in all cohorts, was 1.4% for rupture and 0.2% by subject.
Conclusions: This study demonstrates excellent safety and

Level of Evidence

Editorial Decision date

Breast implant surgery procedures has been shown to continue to be the most advanced technique which, c

Breast Surgery

Ten-Year Safety Data for Eurosilicone's Round and Anatomical Silicone Gel Breast Implants

Franck Duteille, MD; Pierre Perrot, MD; Marie-Hélène Bacheley, MSc; Erin Bell, BSc; and Sharon Stewart, PhD

Aesthetic Surgery Journal
Open Forum
2019, 1-10
© 2019 The American Society for
Aesthetic Plastic Surgery, Inc.
This is an Open Access article distributed under the terms of the
Creative Commons Attribution License (<http://creativecommons.org/licenses/by/4.0/>), which permits unrestricted reuse, distribution, and reproduction in any medium, provided the original work is properly cited.
DOI: 10.1093/asj/ofj012
www.asjopenforum.com

OXFORD
UNIVERSITY PRESS

Downloaded from https://academic.oup.com/asj/advance-article-abstract/doi/10.1093/asj/ofj012 by guest on 04 June 2019

Abstract

Background: Although silicone breast implants have been available for over 60 years, their safety and efficacy continue to be assessed via long-term clinical and vigilance studies. Complications often associated with breast implant surgery include but are not limited to capsular contracture and rupture.
Objective: The authors investigate and evaluate the safety and performance of Eurosilicone's (Eurosilicone SAS, Apt Cedex, France) Crystalline Paragel breast implants at least 10 years postimplantation.
Methods: Nine hundred and ninety-five of Eurosilicone's textured mammary implants were implanted in 526 women undergoing primary (423 patients) and revision surgery (103 patients) at 17 centers throughout France. Complications were recorded at 3 months and annually thereafter for 10 years. Descriptive statistics were used and the Kaplan-Meier method was utilized to analyze key complications.
Results: Seventy-four women (98 implants) experienced capsular contracture across all cohorts. The Kaplan-Meier 10-year cumulative risk of capsular contracture (Baker Grade II/IV) per implant was 11.5% in the primary augmentation cohort and 25.2% in the primary reconstruction cohort. Sixteen implant ruptures were observed by surgeon examination giving a Kaplan-Meier risk of 3.8% per patient and 3.5% per implant. Surgical re-intervention (explantation/exchange) was reported 80 times resulting in a Kaplan-Meier cumulative risk of 13.3% and 31.6% for primary augmentation and primary reconstruction, respectively, per patient. Local complication rates including infection and seroma were low with risk rates of 0.6% and 0.2% by subject.
Conclusions: This multicenter clinical study demonstrates the long-term safety and efficacy profile through 10 years for Eurosilicone round and anatomical silicone gel breast implants.

Level of Evidence: 3

Editorial Decision date: April 10, 2019; online publish-ahead-of-print April 27, 2019.

Since the introduction of silicone breast implants in 1962, there have been great advancements in the development and safety of mammary implants.¹ The most advanced silicone technologies have been incorporated into the latest generation of implants utilizing surface texturing. These breast implants are form-stable, filled with a cohesive silicone gel which increases gel stability, minimizing gel bleed. This combined with innovative surgical techniques has contributed to the popularity of breast augmentation amongst women.² Breast augmentation with implants is one of the leading aesthetic procedures performed, with an estimated 1.5 million prostheses implanted per annum,

globally.³ Even with the introduction of alternative breast augmentation techniques such as lipomodelling, breast

Dr Duteille is the Principal Investigator and Dr Perrot is a Clinical Investigator, Centre des Brûlés, Immeuble Jean Monnet, Centre Hospitalier Universitaire, Nantes, France. Ms Bacheley is a Clinical Trial Monitor, Eurosilicone, Apt Cedex, France. Ms Bell is a Clinical Project Associate and Dr Stewart is a Clinical Project Manager, GC Aesthetics (the parent company of Eurosilicone), Glasgow, UK.

Corresponding Author:
Dr Sharon Stewart, GC Aesthetics, 129 Deenykes View, Westfield Industrial Estate, Cumbernauld, Glasgow G68 9HN, UK
E-mail: sharonstewart@gaesthetics.com

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Garantia GCA Comfort Plus™ Warranty

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Para conhecer melhor a garantia GCA Comfort Plus™ Warranty visite:
gcaesthetics.com/comfort-plus-warranty

* Estudo realizado com implantes mamários da GCA®: Round Collection™ e The Matrix™.

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Referências:

1. Dados dos arquivos, GC Aesthetics®, 2021.
2. Dados dos arquivos, GC Aesthetics®, 2020.
3. Dados dos arquivos, GC Aesthetics®, Countries Sales Report, 2021.
4. Dados dos arquivos, GC Aesthetics®, catálogo GCA®, 2021
5. Duteille F, et al. Ten-Year Safety Data for Eurosilicone's Round and Anatomical Silicone Gel Breast Implants. *Aesthet Surg J Open Forum.* 2019;1(2):ojz012. Published 2019 Apr 27.
6. Spear SL, et al. Allergan Silicone Breast Implant U.S. Core Clinical Study Group. Natrelle round silicone breast implants: Core Study results at 10 years. *Plast Reconstr Surg.* 2014;133(6):1354-1361. doi:10.1097/PRS.0000000000000002.
7. Stevens WG, et al. Ten-year Core Study Data for Sientra's Food and Drug Administration-Approved Round and Shaped Breast Implants with Cohesive Silicone Gel. *Plast Reconstr Surg.* 2018;141(4S Sientra Shaped and Round Cohesive Gel Implants):7S-19S. doi:10.1097/PRS.00000000000004350.
8. Caplin DA, et al. MemoryGel Breast Implants: Final Safety and Efficacy Results after 10 Years of Follow-Up. *Plast Reconstr Surg.* 2021;147(3):556-566. doi:10.1097/PRS.00000000000007635.

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GC Aesthetics® é marca comercial registrada da Global Consolidated Aesthetics Limited.